



Practicon® Starbright™ LED Clip-on Headlight

Instructions

The **Starbright LED Clip-on Headlight** clips onto prescription eyewear and most safety glasses for an intense, focused beam of light (5,382 LUX). Solid-state LED operates for up to 20 hours of continuous use on three AAA batteries included. Non-slip mounting tips protect eyewear from scratches. Strong, double-hinged design weighs less than 1.0 oz. (not including power pack).

ON/OFF switch:

The ON/OFF switch is located on the battery case.

Changing batteries:

1. Remove the small screw located on the rear of the battery case.
2. Apply pressure to loosen cover and slide cover off.
3. Replace batteries. Use AAA only.
4. Slide cover back on until it snaps into place.
5. Replace screw to prevent accidental separation of the battery case.

Using the clip:

Open the clip with thumb and forefinger and apply to eyewear.

Pivoting the light:

Dual pivot points adjust up and down into many positions.

Velcro strap:

A velcro strap has been included for attaching wire to eyewear. You may need to cut the velcro to adjust comfort.

Metal clip:

The metal retaining clip can be applied to battery pack if desired. To use metal clip, remove the cover on back to expose adhesive.

Manufactured for

US Pat. D497,683



PRACTICON

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834

Made in USA • IFU 704250_2