



Practicon® DisposEvac™ Regular Hose Adapters

71-42610 · 3/bag

- **To install:** Carefully cut the existing high volume suction valve quick-disconnect off hose, leaving an even, smooth cut. Insert DisposEvac Adapter into hose, pushing firmly until fully seated.
- Lubricate inner O-ring occasionally with silicone lubricant.

For use with DisposEvac All-in-One Evacuation Tips.

Manufactured for

 **PRACTICON**

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834

Made in USA • Inner Pack 7142610_2