

**5mm Inner  
Diameter**

## **Dr. Schwartz's Cap Trap™ Instructions for Use**

**6mm Inner  
Diameter**

1. Select a Cap Trap with inner diameter slightly smaller than healing abutment outer diameter, and press Cap Trap over abutment. Cap Trap may be cut to length for use with shorter hex drivers.
2. Insert hex driver through Cap Trap and loosen abutment. Remove hex driver.
3. With Cap Trap still gripping abutment, rotate and remove abutment using Cap Trap. Abutment may be replaced using Cap Trap as a handle. Once engaged, remove Cap Trap and retighten with hex driver.
4. Dispose of Cap Trap after use.

**3mm Inner  
Diameter**



Do not reuse. This is  
a single use device.



Not made with  
natural rubber latex.

**4mm Inner  
Diameter**

Practicon, Inc. • 1-800-959-9505