

Practicon® Lock-On™ Saliva Ejector Valve Tips

Instructions

Components Included:

- (3) Lock-On SE Valve Tips
- (3) Lock-On SE Valve Tips
Viton® Rings
- (3) Spare Lock-On SE Valve
Tips Viton Rings

To Install a Lock-On Tip:

1. Place the blue tip onto any existing SE valve or adapter (Figure 1). For best results, pre-clean the valve to remove any oils or lubricants before placing.
2. Push the Viton ring onto the tip and roll it into the groove so that it is firmly seated (Figures 2-3).
3. Pull on the tip to ensure proper installation. In case of tip separation, repeat steps 1-2.

The Lock-On SE Valve Tip and Viton O-rings can remain in place and be sterilized along with SE valve bodies using normal sterilization methods.

Optional Accessories:

71-36910 Additional Lock-On SE Valve Tips Viton Rings.

25 per package.



Figure 1



Figure 2



Figure 3