



Practicon® Denta-Sleeve™ 2 Disinfecting Solution Delivery Sleeves

70-85418 • 100/bag

- For handpieces, curing lights, intraoral cameras, A-W syringes, and other heat sensitive instruments.
- Provide complete contact of surface disinfectants.
- Keep sterilized instruments covered until use.
- Non-sterile. One expandable size.
- 78% nylon/22% spandex.



Read instructions
before use.



Non-sterile



Do not reuse. This is
a single use device.



Not made with
natural rubber latex.



Instructions for Use

1. Place a Denta-Sleeve 2 into disinfecting solution. Denta-Sleeve 2 withstands most disinfecting solutions and is autoclavable to 275°F (135°C).
2. Place the saturated sleeve onto a clean instrument ready for disinfection by inserting instrument into the black drawstring-end. Stretch Denta-Sleeve 2 to cover the entire instrument length.
3. Pull drawstrings at top and bottom to tighten and enclose Denta-Sleeve 2 around the instrument. Excess drawstring may be trimmed away, if desired.
4. Return the instrument to the delivery system or place instrument on a clean, dry surface and allow adequate contact time for proper surface disinfection.
5. After disinfection is complete, discard used sleeve. Rinse and dry instrument.



Caution:

Follow specific disinfectant manufacturer's directions regarding contact time, material compatibility, skin exposure, and waste disposal.

Use of Denta-Sleeve 2 is a disinfection process, not a cleaning or sterilizing process. Proper use of surface disinfectants is the sole responsibility of the user.

Denta-Sleeve 2 makes NO claim and is NOT designed to maintain the sterility of contents inside the sleeve after heat sterilization without prior enclosure inside sterilization pouches or autoclave wraps designed and indicated for that purpose.

Manufactured for

PRACTICON

PRACTICAL INNOVATIONS FOR DENTISTRY
1-800-959-9505 • Fax 1-800-278-0885
practicon.com • Greenville, NC 27834
Made in USA • Inner Pack 7085418_1



723896026526