

VacuLUX

SINGLE-USE MOUTHPIECE

SKU: SVLUXP, SVLUXS, SVLUXM, SVLUXL

USER GUIDE



INSTRUCTIONS FOR USE:

Intended Use

The VacuLUX Mouthpiece is intended for use by dental professionals for isolation, bite block, cheek retractor, tongue guard and high-volume evacuation.

Device Description:

Our single-use mouthpieces are manufactured in the USA and available in a range of sizes to ensure a comfortable fit. Made from a soft, flexible polymer, these non-sterile mouthpieces are free of BPA, phthalates, and natural rubber latex.

Selecting the Mouthpiece:

To select the correct mouthpiece size, measure the patient's interincisal opening. If between sizes, we recommend selecting the smaller size.

Approximate Mouthpiece Size Guide by Interincisal Opening Measurements:

Pediatric (SVLUXP): < 30 mm
Small (SVLUXS): 30 mm
Medium (SVLUXM): 30-45 mm
Large (SVLUXL): > 45 mm



Attaching the Mouthpiece:

Slide the Mouthpiece onto the device until the locking post(s) engage. To ease attachment, use water or mouthpiece lubricant to lubricate the inside surface of the Mouthpiece sleeve.

Placing the Mouthpiece:

Moisten the Mouthpiece and the patient's lips with water prior to placing the Mouthpiece within the oral cavity.

1. Hold the sides of the isolation adapter with your index finger and thumb. With your other hand, fold the cheek retractor onto the tongue guard. Then fold the outside edges of the mouthpiece towards the center.
2. Gently insert the mouthpiece, lining up the bite-block onto occlusal surfaces of the teeth, just distal to the mandibular cuspid. Instruct patient to rest their jaw on the bite block to secure.
3. Place the cheek retractor into the buccal vestibule. Adjust the tongue guard in the lingual vestibule as needed. Move the bite block distally to provide more vertical working room. Reposition the patient's lips as needed.

Disposal Instructions:

VacuLUX Single-Use Mouthpieces are designed for single-use only and must be discarded after each use to prevent cross-contamination.

Do not attempt to sterilize or reuse Single-Use Mouthpieces. Autoclavable VacuLUX mouthpieces are available from Ascentcare Dental Products, and are sold separately (SKU: AVLUXA, AVLUXP, AVLUXS, AVLUXM, AVLUXL).

Immediately after use, place the single-use mouthpiece in an appropriate disposal facility in compliance with local, state and federal regulations.

Warnings

Single-Use VacuLUX Mouthpieces are single-use only. Re-use can result in the transmission of infectious pathogens. Mouthpieces must be discarded after use. Single-Use VacuLUX Mouthpieces will melt at sterilization temperatures and cannot be safely re-sterilized after use.

Do not use if the individual mouthpiece packaging has been damaged or opened. Non-Sterile mouthpieces are individually packaged.

Be sure to use established protocols and patient monitoring when using the Mouthpiece in conjunction with sedation dentistry. Patient management by a certified anesthesiologist is strongly recommended.

Precautions

When performing dental procedures requiring high levels of blood evacuation, it is recommended that the Mouthpiece be intermittently flushed with water to prevent coagulation in the Mouthpiece to avoid loss of suction and light transmission.

Instructions for use may also be found online at:
AscentcareDental.com/product-instructions

Visit our website to learn more about the VacuLUX mouthpiece:
www.AscentcareDental.com

For customer support:

email: support@ascentcaredental.com
call: (888) 822-8589

Ascentcare Dental Products, Inc.
335 North Griffin Street
Grand Haven, MI 49417

